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Instructions for use

Please read this information carefully because it will help you to use HYLO DUAL INTENSE® properly.

HYLO DUAL INTENSE®

Sodium hyaluronate and ectoine

For intensive therapeutic lubrication of dry eyes and stabilisation of the tear film. Relieves inflammatory symptoms such as itching and burning.

Sterile, phosphate-free eye drops without preservatives.

What is HYLO DUAL INTENSE®?

HYLO DUAL INTENSE® is a sterile preservative-free solution for ocular use.

Composition:

1 ml of HYLO DUAL INTENSE® contains 2 mg sodium hyaluronate, 20 mg ectoine, boric acid, borax and water for injections.

What is HYLO DUAL INTENSE® used for?

HYLO DUAL INTENSE® is for the intensive therapeutic lubrication of dry eyes and stabilisation of the tear film. Relieves inflammatory symptoms such as itching and burning.

When should HYLO DUAL INTENSE® be used?

Burning, itching, tearing, foreign body sensation or the sensation of dry eyes are symptoms of an irritated eye. This is often an indication that the eye is not sufficiently provided with humidity.

In these cases, HYLO DUAL INTENSE® is advantageous because of the dual mode of action of sodium hyaluronate and ectoine.

HYLO DUAL INTENSE® contains the sodium salt of hyaluronic acid, a natural substance found in many organs of the human body including the eye.


Its particular physical property is that it forms a regular, stable and long lasting, lubricating and moisturising film on the eye surface that is not easily rinsed off. This so-called bioadhesive property ensures that the cornea and conjunctiva are protected, enhancing the effect of the physiologic tears so the eyelid spreads easily over the surface of the eye. HYLO DUAL INTENSE® contains a higher amount of sodium hyaluronate which makes it more viscous. This provides the eye with an intensive, soothing and long-lasting moisturising film. Persistent or more severe symptoms are relieved and the eye is protected against further irritation.

The other component is Ectoine. Ectoine is a natural substance formed by microorganisms to protect themselves from extreme environments (e. g. salt lakes). Its specific properties are used in HYLO DUAL INTENSE®. Ectoine increases the binding of water to the cells of the ocular surface to form a physiologic barrier on the conjunctiva. Simultaneously, ectoine stabilizes the lipid layer of the tear film that protects against excessive evaporation of tears.

The interaction of sodium hyaluronate and ectoine provides intensive long lasting lubrication of the ocular surface and protects against evaporation of tears. The typical symptoms itching and burning that occur as inflammatory reactions to environmentally triggered irritations are relieved by HYLO DUAL INTENSE®.

How should HYLO DUAL INTENSE® be applied?


Please do not use HYLO DUAL INTENSE® if the seal on the folding carton is broken or damaged before using for the first time (☹).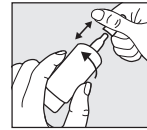

Step 1:


Remove cap before use.

To do this, hold the cap above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement. Before the first application please turn the HYLO DUAL INTENSE® bottle with its tip down and press the bottom of the bottle repeatedly until the first drop appears at the nozzle. Now the bottle is ready for further use.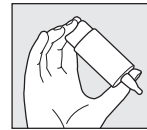


Step 2:

Hold the bottle upside down and place your thumb on the shoulder of the bottle and the other fingers on the base of the bottle.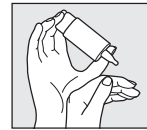


Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.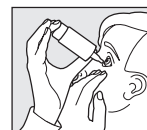

Step 4:

With your head leaning back slightly, gently pull down your lower eyelid. Press the base of the bottle vigorously and quickly. This activates the mechanism to release one drop. Because of the special COMOD® valve system, size and speed of each drop remain constant regardless of the pressure applied to the base of the bottle. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

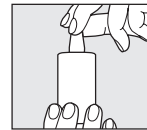Step 5:

Replace cap immediately after use. Ensure that the dropper tip is dry. During use, please avoid any contact between the tip of the bottle and your eye or your skin.

Each bottle releases 10 ml of solution which is the equivalent of approximately 300 drops. For technical reasons a small amount will remain in the bottle.

HYLO DUAL INTENSE® can be used for an unlimited period of time.

HYLO DUAL INTENSE® is only for you, even people with similar symptoms should not use your eye drops.

What else do you need to know?

If you use any other eye drops there should be at least 30 minutes between the applications, and HYLO DUAL INTENSE® should always be used last. Eye ointments should not be administered concomitantly with HYLO DUAL INTENSE®.

Because it is preservative-free HYLO DUAL INTENSE® is well tolerated even when used over a longer period of time. In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately when the use of HYLO DUAL INTENSE® was discontinued.

Please note that temporary blurred vision is possible after application of HYLO DUAL INTENSE® due to the viscosity of the eye drops solution. For the same reason HYLO DUAL INTENSE® should not be used while wearing contact lenses.

There are no controlled clinical data about the use of HYLO DUAL INTENSE® in children, or during pregnancy and lactation. Based on existing evidence HYLO DUAL INTENSE® can be used in children older than 6 years, or during pregnancy and lactation.

How often and for how long can you use HYLO DUAL INTENSE®?

HYLO DUAL INTENSE® should be used according to this instruction for use or the instructions given by your doctor. Generally, adults and children 6 years and older instil one drop of HYLO DUAL INTENSE® three times a day into the conjunctival sac of each eye. If you apply HYLO DUAL INTENSE® more frequently (e.g. more than 10 times per day) or suffer from more severe complaints you should consult an ophthalmologist regularly.

During pregnancy and lactation as well as in children younger than 12 years HYLO DUAL INTENSE® should not be used more than 10 times per day.

HYLO DUAL INTENSE® is suitable for long-term treatment. However – as in all persisting symptoms – you should consult your ophthalmologist if your complaints do not improve e. g. within two weeks.

When should you not use HYLO DUAL INTENSE®?

HYLO DUAL INTENSE® should not be used if you are hypersensitive to one of the ingredients and in children younger than 6 years. HYLO DUAL INTENSE® should not be used while wearing contact lenses.

What you need to know about storing HYLO DUAL INTENSE®?

Do not store above 25 °C (↘^{25 °C}).

HYLO DUAL INTENSE® can be used for 6 months after opening.

Do not use beyond expiry date (📅).

Keep out of the reach of children.

The following package sizes are available:

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Medical device

STERILE A

Your opinion about our product is relevant for us!
We are looking forward to get your feedback via mail or
via www.ursapharm.info/hylo_dual_intense.

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URSAPHARM Arzneimittel GmbH,